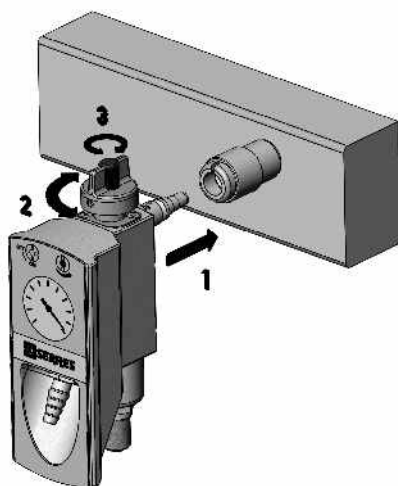
serres

Q-Vac imuejктору
Q-Vac suction ejector
Q-Vac sugejktor
Q-Vac-Gerät erzeugt

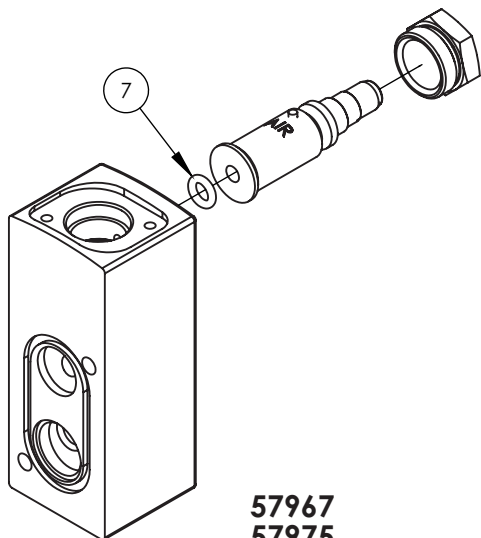
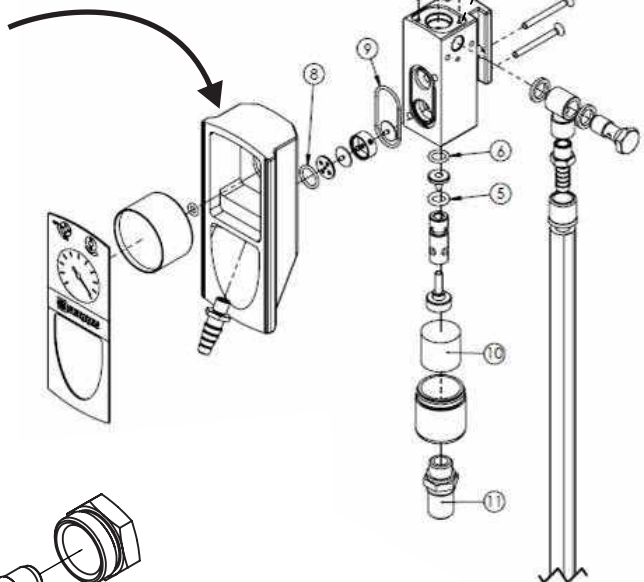
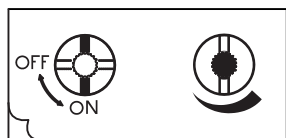
2 Serres Q-Vac 57966, 57967

4 Serres Q-Vac 57964

6 Serres Q-Vac 57977

8 Serres Q-Vac Huolto-ohje
Instruccions for maintenance
Vedlikeholdsinstruksjon
Service instruksjon
Wartungsanleitung

57966
57974

57967
57975

Huoltosarja / Service pack
Servicesats/ Vedlikeholdsserien / Wartungsset

1	o-ring 8,73x1,78	13729	2
2	o-ring 2,57x1,78	13704	1
3	o-ring 12,0x1,5	13730	1
4	o-ring 4,0x1,2	13747	1
5	o-ring 7,3x2,4	13700	1
6	o-ring 7,66x1,78	13701	1
7	o-ring 4,47x1,78	13702	1
8	o-ring 11.1x1,6	13703	1
9	o-ring 28,5x1,6	13732	1
10		13713	1
11		13734	1

FI

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GB

Serres Q-Vac**57966, 57974 rail mounting****57967, 57975 quick-connect coupling**

Suction ejector to the hospital's pressurized-air supply.

The pressure within this supply shall be min 5.2 bar, max 7 bar.

Serres Q-Vac is meant for professional use and the device should be used for the purpose advised by the manufacturer.

Instructions for Use:

1. Connect Q-Vac to the hospital's pressurized-air supply.
2. Start by turning the switch counter-clockwise (90°) and stop by turning the switch clockwise (90°).
3. Using the adjuster, adjust a suitable negative pressure, turning clockwise will produce a lower negative pressure, and turning counter-clockwise will produce a higher negative pressure.

The negative pressure can be read from the negative-pressure gauge.

Storage: Serres ejectors are stored in room temperature.

Boxes should be protected from humidity, dirt and dust.

Cleaning: When necessary, the parts of Q-Vac are cleaned by wiping them with a cleaning and disinfectant agent designed for cleaning hospital equipment.

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GB

Serres Special Sound Absorber with Maintenance Series 57964

Application:

The Serres Special Sound Absorber is an accessory intended for the Serres Q-Vac ejector: if the operational environment requires less noise and a reduction in suction power is no obstacle, a Serres Special Sound Absorber can replace what is being used as a substitute in Serres Q-Vac suction ejectors 57965, 57966, 57967, 57972, 57973, 57974 and 57975. Instructions for the Serres Q-Vac suction injector are supplied with products 57965, 57966, 57967, 57972, 57973, 57974 and 57975.

Installation:

Remove the current sound absorber and install the Special Sound Absorber. A fork spanner with a 17mm width across flats is required for installation. The torque is approximately 6 Nm.

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GB Serres Q-Vac Suction Ejector Degassing Pipe Nipple 57977

Application:

The Serres degassing pipe nipple is an accessory to the Serres Q-Vac Suction Injector. Through the use of the degassing pipe nipple and its connected hose, odorous gases can be directed away from the operating area in a controlled manner.

The degassing pipe nipple can be replaced in Serres Q-Vac Suction Ejectors (57965, 57966, 57967, 57972, 57973, 57974, 57975). Instructions for the Serres Q-Vac Suction Ejector are supplied with products 57965, 57966, 57967, 57972, 57973, 57974 and 57975.

Installation:

Detach the sound absorber and the sound absorber base from the ejector. Twist the degassing pipe nipple into the ejector base and tighten it with a fork spanner (width across flats 10). In addition to this, a hose suitable for degassing is required, by which the gas is directed away from the ejector to the area desired. NOTE: Serres Oy does not supply the hose with the degassing pipe nipple included.

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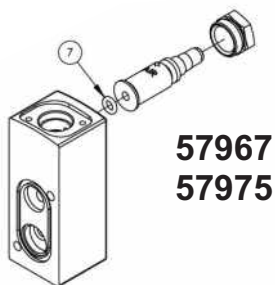
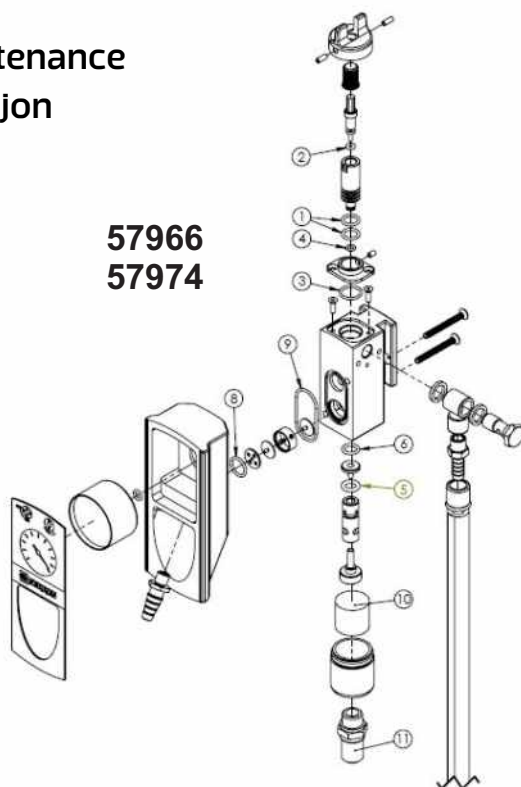
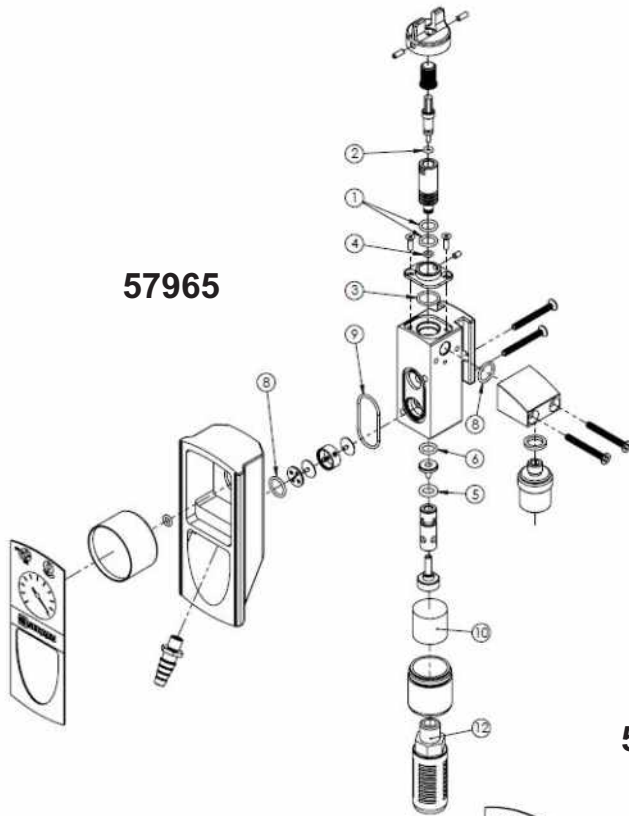
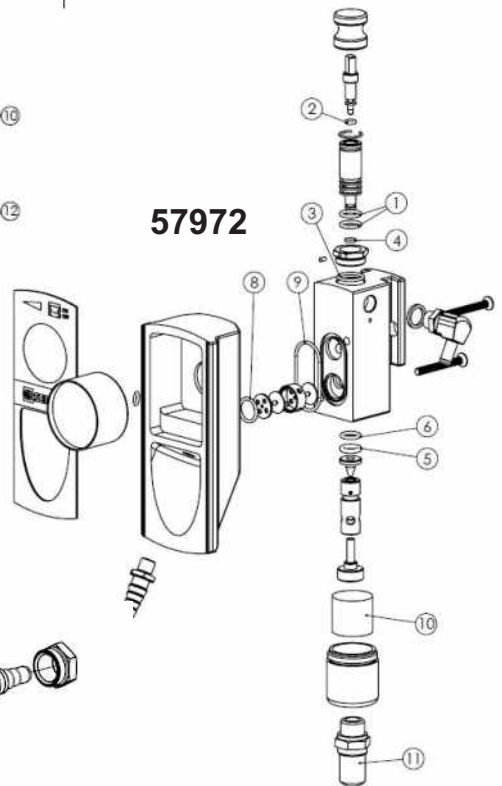
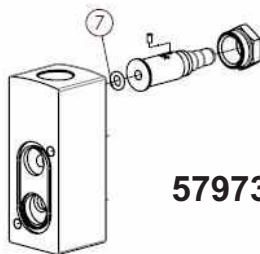
GB

Instructions for Maintenance:

Only persons trained in maintaining hospital equipment are allowed to perform maintenance on this device.

The device is provided with a guard against back-flow. If in connection with the suction process impurities enter the equipment, the device must entirely be taken apart and all parts cleaned with a disinfectant. Parts included in the Service Pack shall be replaced with new ones. The use of an autoclave is prohibited. No parts other than those delivered by the manufacturer may be used. The use of a malfunctioning device is unconditionally prohibited.

The Service Pack is available from the Supplier. When ordering a Service Pack, please check the serial number and model on the rating plate of the device.

Ejector types 57965, 57966, 57967 ja 57975
While assembling the switch two fixing screws must be tightened evenly to enable correct switch movement.

After maintenance, the operation of the device shall be tested:

- air-pressure set at 5,2 bar +/- 0.2 bar
- when setting the negative pressure at maximum, Q-Vac shall reach the following values:
 - negative pressure > 80 kPa
 - suction flow > 27 l/min
 - the quick-closing valve cuts suction
 - the negative-pressure adjuster is functioning
 - the pressure gauge is functioning

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