

serres

Serres Saga quick installation guide



Serres Saga quick installation guide

1/3. Preparing the device

1.
Connect the vacuum hose to the VAC MAIN inlet. Connect a second vacuum hose to the VAC 2 if two channels are in use simultaneously. If not in use, make sure that the VAC 2 inlet is capped.



2.
Power ON the device. As soon as the power is on, the self-check sequence will start.



3.
Install 1 to 6 liners. **DO NOT OPEN** the tape and unfold the liner; liners will automatically unfold with vacuum. Close the canister lid and ensure that you secure the locking latches with a clicking sound. Connect the liner tube to the corresponding liner port inlet of the cart.



4.
Check that there is solidifying agent left above the index line; otherwise install a new cartridge. **DO NOT REMOVE** the protective sealing from the new cartridge before installing it. **ALWAYS LEAVE** a cartridge attached to a container.




5.
Install a new manifold for each patient. Install a second manifold if both channels are in use simultaneously. Installing the manifold triggers the unfolding of the installed liners.



6.
Hang 3 or 5 litre fluid bags but not more than 10 litre per pole. Use strain relief for irrigation tubes. Raise the fluid bag to the desired height.



7.
Check the settings that irrigation and deficit monitoring are activated if necessary.

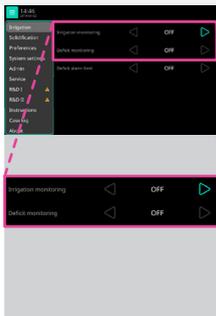


8.
Test the suction by placing a finger onto an opened suction port. Install 1 to 3 suction tube(s) per manifold.




Serres Saga quick installation guide

2/3. During the procedure


- 1.**
Reset values to zero, if necessary (only available up to 500ml of collected liquid).

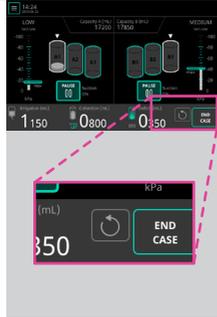


- 2.**
Select the suction limit by moving the soft key to the desired suction limit.

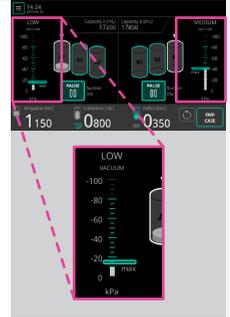

- 3.**
Start suction by tapping START/STOP in each channel to be used.

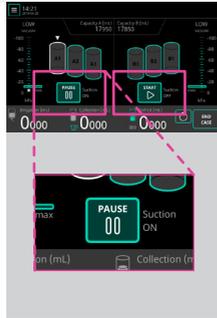

- 4.**
Monitor liquid volumes, remaining collection capacity and liquid deficit.

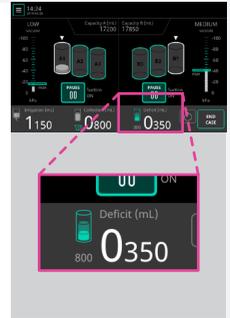

**LIQUID DEFICIT =
IRRIGATION - COLLECTION**
If collection is bigger than irrigation, the liquid deficit is dashed. Liquid deficit is not a diagnostic tool.

- 5.**
Add collection capacity by changing the channel, replacing liners individually or installing a new cart.




- 6.**
Add irrigation fluid by hanging the new bag on one of the hooks. Max 10 litres per pole. DO NOT REMOVE any bags (empty or full) once placed on the poles to secure an accurate irrigation volume.



Serres Saga quick installation guide

3/3. After the procedure

- 1.**
Tap the END CASE key to end a procedure. Solidifying agent is dispensed automatically if this option is enabled.

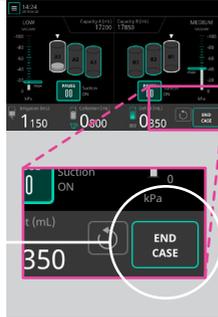

- 2.**
Dispensing of solidifying agent in liners can also be triggered from the Solidification Menu.



- DO NOT REMOVE** the manifold before solidification or before ending the case.

- 3.**
Remove used manifold(s) by lifting the manifold facing downward.



- 4.**
Remove irrigation fluid bags.

- DO NOT REMOVE** fluids bags before ending the case.



- 5.**
Remove liners if full or if used at the end of the day. **DO NOT REUSE** liners removed from the device.



- 6.**
Set the device into standby mode or shutdown the device at the end of the day.



- 7.**
Disconnect vacuum hose(s) and the power cord if necessary.



- 8.**
Clean and disinfect the device.

