Serres

REF 6004479

EN Serres® Collection Pouch Valve

Collection Pouch Valve is intended for use as an accessory to the Serres suction bag system for the suction of liquid from surgical collection pouch. Serres Collection Pouch Valve lets liquid, but not air, pass through from the pouch, to maintain suction efficiency on other devices connected to the same suction system.

CAUTIONS: Use Serres Collection Pouch Valve only with a collection pouch with a sieve as tissue or bone chips can obstruct the valve. Connect Serres Collection Pouch Valve directly to the collection pouch.

CONSULT INSTRUCTIONS FOR USE. Installation: Connect suction tube to the suction system. Connect other end of the suction tube to the Collection Pouch Valve. Connect Collection Pouch Valve directly to the collection pouch. The system is ready for use.

DISPOSAL: Detach the suction tube from the suction system and put the suction tube into the collection pouch to avoid spillage from the tube. Used Collection Pouch Valve may contain biohazardous fluid waste and must be disposed of according to local or hospital-specific regulations.

DO NOT REUSE Collection Pouch Valve is for single-patient use only and must be replaced between patients. Reuse of the Collection Pouch Valve increases risk of cross-contamination between patients.

USE-BY-DATE Collection Pouch Valve use-by date is set up at 5 years after the manufacturing date.

MD MEDICAL DEVICE

Reporting of serious incidents Any serious incident related to the use of this product, should be reported to both the manufacturer and the health authority/competent authority where the product is installed.

EU/HEP Swiss Authorised representative

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